

When will your report be available?

Your test report will be available in 2-3 business days upon receipt of your sample at the lab. The results will be provided through the Camtech Health App under “Results”.

Notes & precautions

1. Do NOT place the swabs back into the plastic wrapper.
2. Insufficient specimen may require recollection and will delay test results.
3. Ensure that you swab only the inside of the cheeks, avoiding the gum area, as collecting too much saliva may affect the results.
4. Collection swab may be a choking hazard to small children.
5. Store and transport the kit at room temperature (15°C - 30°C). Keep it away from direct sunlight and heat sources.

Checklist

- ✓ Complete all the requested information on the two DNA sample envelopes for the Alleged Father and Child.
- ✓ Once the cheek swabs are taken, please post the samples within 24 hours.

Technical Assistance

If you have any questions, please contact:

info@camtech.org



Camtech Paternity DNA Test

INSTRUCTIONS FOR USE FOR HOME USE

Kit contents:


- 2 swab packs for the child (4 swabs in total)
- 2 swab packs for the alleged father (4 swabs in total)
- 2 sample envelopes
- 1 resealable plastic bag
- 1 mail envelope
- 1 set of instructions





STEP 1

Scan this QR code to download the Camtech Health App. Sign up for a new account, if you don't have one yet.

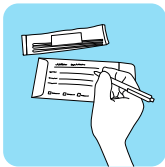


STEP 2

Remove the sample envelopes from the plastic bag. Mark one of the envelopes for the "Alleged Father" and the other envelope for the "Child" and fill in the requested details.



STEP 3

Click the "Start Test" button and scan the barcode on any of the sample envelopes. Review the Terms of Service and Privacy Policy in the app. Follow the instructions on the screen or the instructions below.

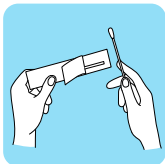

STEP 4

Wash your hands thoroughly. Open the plastic wrapper at the end opposite from the cotton tip. Remove one swab from the package, taking care not to touch the end with the cotton tip.




STEP 5

Open the mouth wide, insert swab into the mouth and rub it firmly back and forth and up and down on the inside of one cheek. It is important to collect enough samples on the entire swab, so be sure to rotate the swab while rubbing.




STEP 6

Once the sample is collected, leave the swab to air dry for 15 minutes while doing the next steps. You may place the swab on a holder as long the tip is not touching anything.

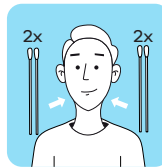

STEP 7

Repeat the collection process (steps 5 and 6) for the other swabs. Use a total of 2 swabs on the right cheek and 2 swabs on the left cheek for each tested person.

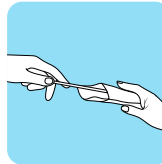


STEP 8

Place the swabs into the sample envelope labeled for that person. Ensure that the cotton tip of the swab is facing inside the envelope and not towards the open end where the envelope has to be closed.



STEP 9

After all 4 swab packs have been used, discard the plastic wrappers and make sure the flaps of the sample envelopes are tucked in and labeled for each person. Put the envelopes back into the resealable plastic bag and close it.

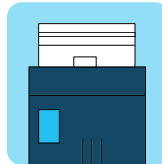

STEP 10

Place the plastic bag with the sample envelopes back in the mail envelope and seal it. Peel off your home address label from the front. The return address is already affixed to the back of the envelope. Drop this mail envelope off at the nearest Singpost box.

